

Bio-Tek ELx50 Optimal Settings for VMRD's Equine Infectious Anemia Virus Antibody Test Kit, ELISA v2

Below is a listing of the optimal settings for running VMRD's EIAV Antibody Test Kit, ELISA (catalog number 5515.01-1).

EIAV 1st (2nd)

Method	No. of Cycles	1 (4)	
	Wash Format	Plate	
	Soak Shake	No	
Disp	Dispense Vol	250 µl	
	Flow Rate	4	450 µl/sec
	Disp Height	120	15.24 mm
	Horizon Disp Pos	0	0.000 mm
	Bottom Wash 1st	No	
	Bottom Wash Disp Vol	N/A	
	Bottom Wash Flow Rate	N/A	
	Bottom Wash Disp Height	N/A	
	Bottom Wash Horiz Pos	N/A	
	Prime Before Start	No	
	Prime Vol	N/A	
	Prime Flow Rate	N/A	
Aspir			
	Aspirate Height	024	3.048 mm
	Horiz Aspirate Pos	-19	-0.869 mm
	Aspirate Rate	3	5.8 mm/sec
	Aspirate Delay	0	
	Crosswise Aspirate	No	
	Crosswise On	N/A	
	Crosswise Height	N/A	
	Crosswise Horiz Pos	N/A	
	Final Aspirate	Yes	
Final Aspirate Delay	0		