



VMRD, Inc.

PO Box 502, Pullman, WA 99163 USA

Telephone: + 1 (509) 334-5815

Fax: + 1 (509) 332-5356

E-mail: vtech@vmrd.com

Web site: <http://www.vmrd.com>

Certificate of Analysis

CANINE COOMBS' REAGENT

CATALOG NO.: 392-5

VOLUME: 5 ml

LOT: V02040-050704

EXPIRATION DATE: 7 May 2004

INTRODUCTION: The canine Coombs' test, also called direct antiglobulin test, is designed to detect immune-mediated erythrocyte destruction which occurs in autoimmune hemolytic anemia, canine systemic lupus erythematosus, and in some cases with infections and neoplastic disorders. Hemolysis in these diseases is caused by the erythrocytes being coated with antibody (IgG, IgM) and/or complement components (C3). Coated erythrocytes are either lysed in the bloodstream or removed by phagocytes.

The Coombs' reagent is an antiserum to canine IgG, IgM, and C3 prepared in goats. After obtaining the antiserum from the goat, complement is inactivated at 56°C for 30 minutes and then the antiserum is absorbed repeatedly with washed normal canine erythrocytes. These treatments ensure that the Coombs' reagent will not react with normal canine erythrocytes. However, canine erythrocytes that are coated with IgG, IgM, and/or C3 will be agglutinated by the Coombs' reagent that still contains antibodies to canine IgG, IgM, and C3.

INDICATIONS FOR TEST: All dogs with anemia (including that caused by intravascular and extravascular hemolysis) of unknown origin are reasonable candidates for evaluation by Coombs' testing.

PROCEDURE:

- A. Erythrocytes for testing can be obtained a number of ways and are listed in order of preference:
1. Blood collected in ethylenediamine tetraacetic acid (EDTA).
 2. Blood collected in heparin.
 3. Erythrocytes teased from clotted blood.

Note: Whenever possible, blood from a healthy non-anemic dog should be evaluated along with blood from the anemic dog. Blood from the normal dog will serve as a negative control.

FOR RESEARCH PURPOSES ONLY. NOT FOR DIAGNOSTIC OR THERAPEUTIC PROCEDURES.

WARRANTY: VMRD, Inc. warrants that this product is as described in the quantity and contents stated on the label at the time of delivery to the customer. NO OTHER WARRANTIES, EXPRESS OR IMPLIED, ARE MADE BEYOND THE LABEL DESCRIPTION, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR USE. Remedy is limited to replacement of the product or refund of the purchase price. VMRD, Inc. is not liable for property damage, personal injury, or economic loss caused by the product. The information listed in this information sheet is provided for reference only, and should not be substituted for the user's own incoming material quality control.

B. Washing of erythrocytes.

1. Centrifuge blood (standard tabletop centrifuge for 5 minutes at room temperature).
2. Remove 0.1 ml of packed erythrocytes and add to 4.9 ml phosphate buffered saline (PBS) or normal saline solution. (**NOTE:** Other solutions may influence results.)
3. Mix the erythrocytes and PBS. Centrifuge the mixture as above and remove the supernatant. Resuspend the erythrocyte pellet in 4.9 ml of PBS.
4. Repeat the washing procedure in the previous step three more times. This provides for four washings of the erythrocytes.
5. At the end of the last wash remove the supernatant and resuspend the pellet in 4.9 ml of PBS. This provides a 2% suspension of erythrocytes.

C. Dilution of the Coombs' reagent.

1. Label four test tubes (12 x 75 mm) 1, 2, 3, 4 consecutively.
2. Add 0.1 ml PBS to all four tubes.
3. Add 0.1 ml of Coombs' reagent to tube 1, mix well and transfer 0.1 ml of this mixture to tube 2. Mix tube 2 well and then transfer 0.1 ml to tube 3. Mix tube 3 well, then remove and discard 0.1 ml.
4. At the end of this process, tube 1 should contain 0.1 ml of a 1:2 dilution of the Coombs' reagent, tube 2 a 1:4 dilution, and tube 3 a 1:8 dilution. Tube 4 should contain only PBS.
5. Steps C-1 to C-4 should be repeated for each sample to be tested, including the negative control.

D. Coombs' test.

1. Add 0.1 ml of washed resuspended erythrocytes from the dog to be tested to tubes 1 through 4. Gently mix.
2. Incubate for 30 minutes at 37°C.
3. Centrifuge for 1 minute.
4. To resuspend the erythrocytes for examination and to dissociate any nonspecific agglutination, hold each tube at a 45° angle and tap firmly on a table top 15 times just prior to step 5.
5. Evaluate the contents of each tube by placing a small amount of the solution on a slide and viewing with a microscope (100X magnification is suitable).

E. Test interpretation.

Negative—erythrocytes are not clumped or agglutinated.

Positive—There are clumps and large aggregates of erythrocytes. The clumps should not be present in the control cells. Occasional clumps (3 or 4 per slide) may occur in test and control erythrocytes and should be disregarded. Hemolysis should not be considered a positive reaction.

PRECAUTIONS: Store at 4°C and use prior to expiration date noted on this sheet. Use the reagent at the dilutions described in the procedure to avoid nonspecific and prozone effects.

REFERENCES:

- Quimby, F.W., *et al.* Efficacy of immunoserodiagnostic procedures in the recognition of canine immunological diseases. *Am. J. Vet. Res.* 41:1662-1666 (1980).
- Slappendel, R. The diagnostic significance of the direct antiglobulin test (DAT) in anemic dogs. *Vet. Immunol. Immunopathol.* 1:49-59 (1979).
- Halliwell, R.E.W. Autoimmune disease in the dog. *Adv. Vet. Sci. Comp. Med.* 22:221-263 (1978).