



VMRD, Inc.

PO Box 502, Pullman, WA 99163 USA

Telephone: + 1 (509) 334-5815

Fax: + 1 (509) 332-5356

E-mail: vtech@vmrd.com

Web site: <http://www.vmrd.com>

Certificate of Analysis

EQUINE COOMBS' REAGENT

CATALOG NO.: 492-2

VOLUME: 2 ml

LOT: V98096-022105

EXPIRATION: 21 February 2005

INTRODUCTION: The equine Coombs' test, also called direct antiglobulin test, is designed to detect immune-mediated erythrocyte destruction which occurs in autoimmune hemolytic anemia, in some cases with infectious and neoplastic disorders, and in neonatal isoerythrolysis. Hemolysis in these diseases is caused by the erythrocytes being coated with antibody (IgG, IgM) and/or complement components (C3). Coated erythrocytes are either lysed in the bloodstream or removed by phagocytes.

The Coombs' reagent is an antiserum to equine IgG, IgM, and C3 prepared in goats. After obtaining the antiserum from the goat, complement is inactivated at 56°C for 30 minutes and then the antiserum is absorbed repeatedly with washed normal equine erythrocytes. These treatments ensure that the Coombs' reagent will not react with normal equine erythrocytes. However, equine erythrocytes that are coated with IgG, IgM, and/or C3 will be agglutinated by the Coombs' reagent that still contains antibodies to equine IgG, IgM, and C3.

INDICATIONS FOR TEST: All horses with anemia (including that caused by intravascular and extravascular hemolysis) of unknown origin are reasonable candidates for evaluation by Coombs' testing. Foals with neonatal isoerythrolysis are often Coombs' positive.

PROCEDURE:

- A. Erythrocytes for testing can be obtained a number of ways and are listed in order of preference:
1. Blood collected in ethylenediamine tetraacetic acid (EDTA).
 2. Blood collected in heparin.
 3. Erythrocytes teased from clotted blood, being careful to remove clumps.

Note: Whenever possible, blood from a healthy non-anemic horse should be evaluated along with blood from the anemic horse. Blood from the normal horse will serve as a negative control.

FOR RESEARCH PURPOSES ONLY. NOT FOR DIAGNOSTIC OR THERAPEUTIC PROCEDURES.

WARRANTY: VMRD, Inc. warrants that this product is as described in the quantity and contents stated on the label at the time of delivery to the customer. NO OTHER WARRANTIES, EXPRESS OR IMPLIED, ARE MADE BEYOND THE LABEL DESCRIPTION, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR USE. Remedy is limited to replacement of the product or refund of the purchase price. VMRD, Inc. is not liable for property damage, personal injury, or economic loss caused by the product. The information listed in this information sheet is provided for reference only, and should not be substituted for the user's own incoming material quality control.

B. Washing of erythrocytes.

1. Centrifuge blood (1500 x g for 5 minutes at room temperature).
2. Remove 0.1 ml of packed erythrocytes and add to 4.9 ml phosphate buffered saline (PBS) or normal saline solution.
3. Mix the erythrocytes and PBS. Centrifuge the mixture as above and remove the supernatant. Resuspend the erythrocyte pellet in 4.9 ml of PBS.
4. Repeat the washing procedure in the previous step three more times. This provides for four washings of the erythrocytes.
5. At the end of the last wash, remove the supernatant and resuspend the pellet in 4.9 ml of PBS. This provides a 2% suspension of erythrocytes.

C. Dilution of the Coombs' reagent.

1. Label four test tubes (12 x 75 mm) 1, 2, 3, 4 consecutively.
2. Add 0.1 ml PBS to all four tubes.
3. Add 0.1 ml of Coombs' reagent to tube 1, mix well and transfer 0.1 ml of this mixture to tube 2. Mix tube 2 well and then transfer 0.1 ml to tube 3. Mix tube 3 well, then remove and discard 0.1 ml.
4. At the end of this process, tube 1 should contain 0.1 ml of a 1:2 dilution of the Coombs' reagent, tube 2 a 1:4 dilution, and tube 3 a 1:8 dilution. Tube 4 should contain only PBS.
5. Steps C-1 to C-4 should be repeated for each sample to be tested, including the negative control.

D. Coombs' test.

1. Add 0.1 ml of washed resuspended erythrocytes from the horse to be tested to tubes 1 through 4. Gently mix.
2. Incubate for 30 minutes at 37°C.
3. Centrifuge for 1 minute at 1500 x g.
4. To dissociate any nonspecific agglutination, hold each tube at a 45° angle and tap firmly on a table top 15 times just prior to step 5.
5. Evaluate the contents of each tube by placing a small amount of the solution on a slide and viewing with a microscope (100X magnification is suitable).

E. Test interpretation.

Negative—erythrocytes are not clumped or agglutinated.

Positive—There are clumps and large aggregates of erythrocytes. The clumps should not be present in the control cells. Occasional clumps (3 or 4 per slide) may sometimes occur in test and control erythrocytes and should be disregarded. Hemolysis should not be considered a positive reaction. Horses with active equine infectious anemia (EIA) have C3-coated erythrocytes and will produce a positive Coombs' test. Such horses will also have a positive Coggins' test.

PRECAUTIONS: Store at -10°C indefinitely or at 4-8°C and use within 6 months of purchase. Use the reagent at the dilutions described in the procedure to avoid nonspecific and prozone effects.

REFERENCES:

- McGuire, T.C., *et al.* Complement (C3)-coated red blood cells following infection with the virus of equine infectious anemia. *J. Immunology* 103:239-299 (1969).
- Anderson, I.J. Idiopathic autoimmune haemolytic anemia in the horse. *New Zealand Vet. J.* 22:102 (1974).
- Collins, J.D. Autoimmune haemolytic anemia in horse. *Proc. 1st Int. Equine Hematology Symp.*, p. 32 (1975).
- Trommershausen-Smith, A., *et al.* Alloantibodies: Their role in equine neonatal isoerythrolysis. *Proc. 1st Int. Equine Hematology Symp.*, p. 349 (1975).
- Piercy, R.J., *et al.* Erythroid hypoplasia and anemia following administration of recombinant human erythropoietin in two horses. *JAVMA* 212:244-247 (1998).